



**Re: INTERCEPT Blood System™ for Platelets and Plasma  
Product Code INT100: Expanded Disinfectant List for the INTERCEPT Illuminator**

Dear Valued Customer:

In response to your feedback, we are pleased to advise you of an expansion to the approved labeling for the INTERCEPT Illuminator.

**INTERCEPT Illuminator Disinfectant List Expanded to Include Ten New Disinfectants**

Ten new disinfectants have been added to the list of products suitable for use with the INTERCEPT Illuminator. Table 1 specifies the disinfectants tested. Per the table, please note that within the set of new disinfectants, Bacillol AF should **NOT** be used on the illumination tray. Use a soft cloth to apply the disinfectants. After the manufacturer’s recommended contact time, use a soft cloth dampened with water to remove the solution. All other preparation/application parameters should be as per the disinfectant manufacturer’s instructions. Disinfectants were deemed suitable based on material compatibility with illuminator surfaces; Cerus makes no claims about the microbial disinfection efficacy of the products.

**Table 1 Disinfectants Suitable for Use with INTERCEPT Illuminators**

Disinfectant Name and Manufacturer	Suitable for Use?			
	Illumination Tray	Reflective Divider in Tray	Screen and Keypad	External Surface
Freshly Prepared Bleach (10%) (Sodium hypochlorite)	YES	<b>NO</b>	YES	YES
Iodine based disinfectant	YES	YES	<b>NO</b>	YES
LpH se disinfectant	<b>NO</b>	YES	YES	YES
<b>NEW</b>				
Bacillol AF (ready to use) (Bode, Germany)	<b>NO</b>	YES	YES	YES
CaviCide (ready to use) (Metrex, USA)	YES	YES	YES	YES
DAX 70+ (ready to use) (Opus Health Care, Sweden)	YES	YES	YES	YES
Gigasept FF New (5%) (Schulke & Meyr, Germany)	YES	YES	YES	YES
Incidin Plus (2%) (Ecolab, Germany)	YES	YES	YES	YES
Kohrsolin Extra (0.5%) (Bode, Germany)	YES	YES	YES	YES
Mikrobac Tissues (ready to use) (Bode, Germany)	YES	YES	YES	YES
Mikrozid Sensitive Liquid (ready to use) (Schulke & Meyr, Germany)	YES	YES	YES	YES
M-Ytdes (ready to use) (Kemetyl, Sweden)	YES	YES	YES	YES
Terralin Protect (2%) (Schulke & Meyr, Germany)	YES	YES	YES	YES



## **Implementation**

Adoption of new disinfectants should proceed according to the internal processes of your blood establishment and in compliance with local regulations and procedures.

## **Contact information**

If you have any questions concerning this change, please contact Cerus BV Customer Services:

Stationsstraat 79-D  
3811 MH Amersfoort  
The Netherlands

Email: [customer\\_services@cerus.com](mailto:customer_services@cerus.com)  
Phone: +31 33 496 0600  
Fax: +31 33 496 0606

With Kind Regards,

*Cerus Europe BV*